ALC%CONTROL

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*SIMPLE. SAFE. TÜV-TESTED.
SIMPLE. SÛR. CERTIFIÉ TÜV.
EENVOUDIG. ZEKER. TÜV-GETEST.*

www.testme.de

START

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Use approximately 20 minutes after your last alcoholic drink. No smoking or eating immediately before use!

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Take a deep breath! Blow into the mouth piece for approximately 10 seconds (not much pressure needed). Repeat!

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CAUTION: If exhaled air does not escape on the other end of the device or the crystals do not heat up, repeat the test!

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SAFETY WARNINGS:

The breathalyser should be used once only | stored at min. -25°C to max. +40°C | kept out of the reach of children | stored safely and kept dry | used only according to instructions (individual components can be damaging to your health) | discarded unused if broken (danger of cuts from glass)

LIABILITY QUESTIONS:

The test result can only be used as guidance by the user and is no legal evidence.

The user is solely responsible for actions or decisions after using the breathalyser. The manufacturer or merchant of the breathalyser are not liable. After use, discard in the yellow bin.

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Press lids on both ends of the device simultaneously towards the centre (see labelling).

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Your test result will show on the scale after a few seconds!

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CAUTION: If the white crystals are discoloured, the user is under the influence of alcohol and should not drive.

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