Universalbandage

aus recyceltem Kunststoff

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Haagweg 12

65462 Ginsheim-Gustavsburg

Material: 87% Polyester recycelt, 13% Elasthan

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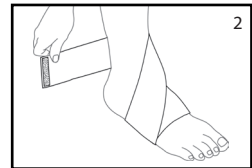
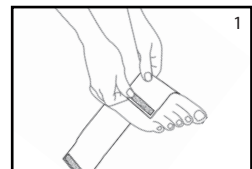
GB | Universal bandage, made from recycled plastic

1 | Delivery

1 x Bandage

Read the instructions for use completely and carefully!

2 | Intended purpose

The bandage supports and stabilizes. Targeted pressure on tendons and muscles can relieve pain and relieve the musculoskeletal system. The aim is to reduce the use of painkillers and increase the patient's mobility.

- supports the muscles in case of joint problems, ligament and muscle weakness
- in case of slight twists and sprain
- in case of overload pain
- in case of tendonitis#

For private use

3 | Contraindications / Not suitable

In the case of the following medical conditions, it is only recommended to wear this article after consultation with your doctor:

- Skin diseases/injuries in the affected part of the body, especially in inflammatory symptoms such as swelling, redness and overheating
- Allergic reactions to the material
- Sensory and movement disorders, e.g. in the case of diabetes
- Open wounds or internal injuries
- For small children or persons who can not be noticeable when the product sits uncomfortably
- Lymphatic runoff disorders or unclear soft tissue swelling even outside the scope of application.

4 | Notes on safe use of the product

- Side effects affecting the whole organism are not yet known. Proper application is required. In case of improper use, product liability is excluded.
- If the bandage is too tight or uncomfortable, remove it immediately!
- Used by the times by private users.
- If your symptoms worsen or you notice unusual changes, consult your doctor immediately.

Do not use: While sleeping

5 | Application

The application of the universal bandage differs slightly depending on the body part. Generally, the joint is always wrapped. (Fig. On the example ankle).

Fig. 1: Place the bandage on the instep and wrap it around the foot. Fig. 2+3: Guide the bandage diagonally across the instep, around the ankle (if necessary several times). Fasten the hook and loop closure anywhere on the bandage. By pulling the strap you can stabilize the joint individually. When fitting the strap, please make sure there are no creases.

6 | Cleaning

Close hook and loop fasteners during washing to avoid damaging other items of laundry. Cleaning is done by household laundry, according to care instructions.

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